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ppa. Wilfried Beyerle
Director Quality Systems

Counterfeit Prime & Bond NT Refill Package

Comparison between a counterfeit Prime & Bond NT Refill and a retain sample. Three complaints were received by customers complaining about rheological properties of the material: PR# 1453020, PR# 1453098, PR# 1457345. All three customers complained Prime&Bond NT EU Refill 60667240, Lot 1511000479, expiring 2018-11.

	Complaint reason
PR# 1453098	one bottle even sealed was leaking
PR# 1457345	the opening hole is too big, material runs out too fast
PR# 1453020	material runs out too fast

The complaints were compared against lot of Prime&Bond NT EU Refill 60667240, lot 1511000987, expiring 2018-11.

Original	Counterfeit
<p><u>Box Design</u></p> <ul style="list-style-type: none"> - box layout corresponds to the authorized sample of an EU version refill - on the box was used an aluminum seal - box version is K107361-05 	<p><u>Box Design</u></p> <ul style="list-style-type: none"> - box layout differs in many ways from actual box layout version. - box layout is comparable to a former version, replaced in 2015-06 - no aluminum label used ➔ box is NOT originally from Dentsply DeTrey GmbH
 <p>The image shows the original Prime & Bond NT Refill box. It is white with a red curved design on the right side. The text 'Prime & Bond NT' is visible at the top. A barcode is located at the bottom, with the lot number 'LOT 1511000987 E XP 2018-11' printed below it.</p>	 <p>The image shows a counterfeit Prime & Bond NT Refill box. It is white with a red curved design on the right side. The text 'Prime & Bond NT' is visible at the top. A barcode is located at the bottom, with the lot number 'LOT 1511000479 2018-11' printed below it. A handwritten number '#1453025' is circled in orange on the box.</p>

17.3.2016

Seite 1 von 7

Counterfeit
Prime & Bond NT Refill Package

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Counterfeit Prime & Bond NT Refill Package

Original (lot)	Counterfeit (item 606.67.240)
<p><u>Box Content – Bottle</u></p> <ul style="list-style-type: none"> - The filled bottle and the label matches to authorized sample - Label layout is K107363-04 → actual version 	<p><u>Box Content – Bottle</u></p> <ul style="list-style-type: none"> - Printed lot number 1511000479 is <u>NOT</u> registered in the Dentsply database - Label layout is K107363-03 → outdated version, no longer in use. - The screw-cap was closed tightly to provide the impression that the bottles were sealed originally, but the seal of all bottles was broken → Bottle is <u>NOT</u> an original bottle in use for Prime&Bond NT, screw cap and dripping insert seem to be original → Bottle label is <u>NOT</u> originally from Dentsply DeTrey GmbH
	
	

17.3.2016

Seite 3 von 7

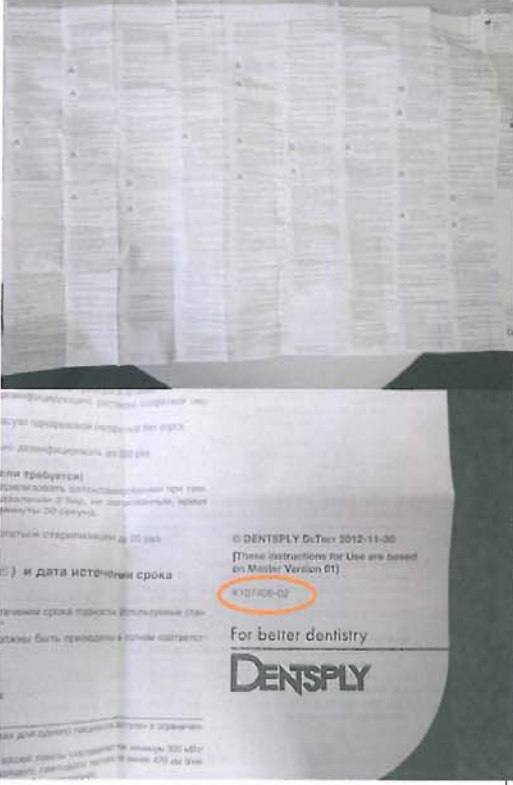
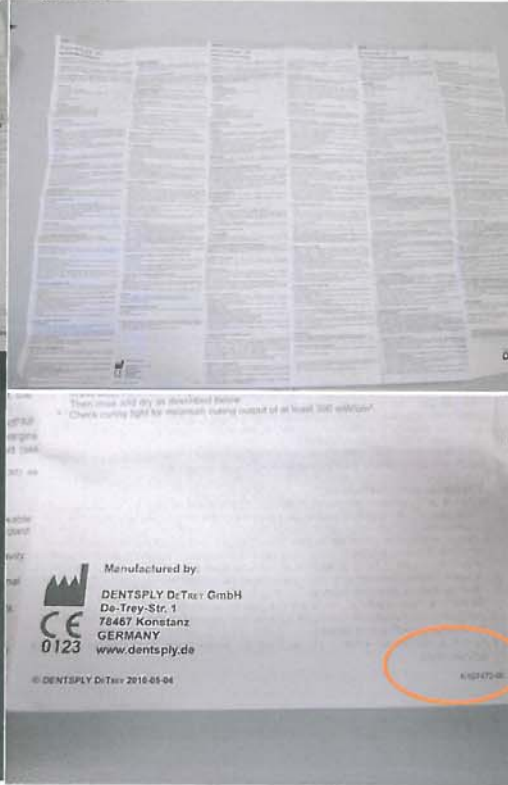
Counterfeit
Prime & Bond NT Refill Package

17.3.2016

Seite 4 von 7

Counterfeit
Prime & Bond NT Refill Package

Original (lot)	Counterfeit (item 606.67.240)
<p>Box Content – IFU</p> <ul style="list-style-type: none"> - K107406: Instruction for use P&B <u>EU Refill</u> 4.5ML 	<p>Box Content – IFU</p> <ul style="list-style-type: none"> - K107472-00: Instruction for use P&B <u>NON-EU Refill</u> 3.5ML → outdated IFU Version, replaced in 2015-10 → IFU NOT matching P&B <u>EU Refill</u> 4.5ML, different IFU (K107406) should have been packed
	

17.3.2016

Seite 5 von 7

Counterfeit
Prime & Bond NT Refill Package

Original (lot)	Counterfeit (item 606.67.240)																					
<p><u>Box Content – Liquid</u></p> <p>- Not applicable</p>	<p><u>Box Content – Liquid</u></p> <p>Testing of ingredients:</p> <p>- HPLC assay of ingredients does <u>NOT</u> match to specifications of Prime&Bond NT</p> <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th style="text-align: center;">Prime&Bond NT major components No.:</th> <th style="text-align: center;">Reference percentage [%]</th> <th style="text-align: center;">Percentage found in complaint [%]</th> </tr> </thead> <tbody> <tr> <td style="text-align: center;">1</td> <td style="text-align: center;">100</td> <td style="text-align: center;">0</td> </tr> <tr> <td style="text-align: center;">2</td> <td style="text-align: center;">100</td> <td style="text-align: center;">124</td> </tr> <tr> <td style="text-align: center;">3</td> <td style="text-align: center;">100</td> <td style="text-align: center;">4101*</td> </tr> <tr> <td style="text-align: center;">4</td> <td style="text-align: center;">100</td> <td style="text-align: center;">240</td> </tr> <tr> <td style="text-align: center;">5</td> <td style="text-align: center;">100</td> <td style="text-align: center;">95</td> </tr> <tr> <td style="text-align: center;">6</td> <td style="text-align: center;">100</td> <td style="text-align: center;">0</td> </tr> </tbody> </table> <p style="text-align: center;">*: Outside the calibration range</p> <p>Testing of adhesion:</p> <p>The data obtained for the test purchase material means for the adhesion to enamel (15,6 MPa) and dentin (1,8 MPa → should be 10 MPa or above) is NOT in a typical range for Prime & Bond NT and does <u>NOT</u> provide a sufficiently strong adhesion of the material.</p> <p>→ The liquid in complained Prime&Bond NT bottles is <u>NOT</u> Prime&Bond NT. It does <u>NOT</u> provide sufficiently strong adhesion to dentin to be a dental bonding.</p>	Prime&Bond NT major components No.:	Reference percentage [%]	Percentage found in complaint [%]	1	100	0	2	100	124	3	100	4101*	4	100	240	5	100	95	6	100	0
Prime&Bond NT major components No.:	Reference percentage [%]	Percentage found in complaint [%]																				
1	100	0																				
2	100	124																				
3	100	4101*																				
4	100	240																				
5	100	95																				
6	100	0																				

17.3.2016

Seite 6 von 7

Counterfeit Prime & Bond NT Refill Package

Conclusion:

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
Item under investigation: Prime&Bond NT EU Refill 60667240, Lot 1511000479, expiring 2018-11.
Reference item: Prime&Bond NT EU Refill 60667240, lot 1511000987, expiring 2018-11.

The differences between the Item under investigation and the reference item are obvious:

- The printed lot number 1511000479 does NOT exist in connection with Prime&Bond NT
- The carton box, the bottle and the bottle label are NOT originally from Dentsply DeTrey GmbH
- The available instruction for use (IFU) P&B NON-EU Refill 3.5ML is NOT matching P&B EU Refill 4.5ML. The packed IFU does NOT contain instruction for use in German language. Furthermore an outdated versions of this IFU have been packed.
- The liquid in complained Prime&Bond NT bottles is NOT Prime&Bond NT.
- It does NOT provide sufficiently strong adhesion to dentin to be a dental bonding → fillings are possible to debond.
- The outer appearance of the counterfeit item Prime&Bond NT EU Refill has been made in a very professional manner. Customers are not expected to distinguish between Dentsply DeTrey GmbH original Prime&Bond NT EU Refill and this counterfeit item.

Medical device has been sold into the market using Dentsply DeTrey GmbH CE-number. Due to the fact a declaration of conformity is not existing, this is violation of medical device law. Therefore a possible health risk for patient user cannot be excluded.

Facing these facts we came to the conclusion that this item is a counterfeit product.

Manager Quality Control

17.3.2016

Seite 7 von 7