# TRUMPF

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## 1.3 Inspection and maintenance intervals

Inspections and maintenance work must be carried out at the specified maintenance intervals.

### HINWEIS

#### Authorized aftersales service

The device must only be serviced by the TRUMPF aftersales service or an authorized partner.

#### Inspection and maintenance of parts procured from third-party manufacturers

Any assemblies or attachments procured from third-party manufacturers must be inspected and serviced in accordance with the specifications of these manufacturers.

Service hotline:

- Phone: ++49 / (0) 180 / 2 25 41 35
- Fax: ++49 / (0) 36 71 / 58 61 75

We recommend you to sign a service contract.

#### Annual inspection **Annual inspection by the user or operator**

Visual inspections and functional tests of the components or functional systems mentioned below must be carried out by the user or operator.

| Component / functional system   | Inspection  |
|---|---|
| Gas outlet points   | In acc. with the manuf.'s                               |
| Power sockets of the electric system  | Visual inspection                                       |
| Brake system  | Functional test   |
| <b>Height-adjustable base support units</b>                                   |   |
| Surface damage<br>Height adjustment mechanism                                 | Visual inspection<br>Functional test                    |
| <b>Sub-pendant</b>  |   |
| Surface damage<br>Lifting operation<br>Adjustment of the anesthetic apparatus | Visual inspection<br>Functional test<br>Functional test |

**Maintenance rates**

**Inspection and maintenance by technical service technicians**

The first maintenance of the ceiling pendants is only due after 5 years.

After this time, the maintenance work specified in the table must be carried out every 2.5 or 5 years.

The inspections and maintenance jobs indicated in the corresponding columns must be performed by an authorized service technician.

Legend:

- No inspection is required during this period
- X Compulsory inspection
- \* Depending on the individual type.

| Bauteil/Funktionssystem                     | Maßnahme                        | 2,5 Jahre | 5 Jahre |
|---|---------------------------------|-----------|---------|
| Gas hoses:<br>Leak tightness                | Functional test                 | —         | X       |
| Gas outlet points                           | In acc. with the manuf.'s spec. |           |         |
| Electrical system:<br>Power sockets         | Visual inspection               | —         | X       |
| Power supply cables                         | Functional test                 | —         | X       |
| Leakage currents                            | Functional test                 | —         | X       |
| Mechanical system:<br>All assemblies        | Visual inspection               | —         | X       |
| Tight fitting                               | Functional test                 | —         | X       |
| Joint bearing                               | Functional test                 | —         | X       |
| Limit stops                                 | Functional test                 | —         | X       |
| Brake system:<br>Braking force              | Functional test                 | X         | X       |
| Leak tightness of the hoses*                | Functional test                 | —         | X       |
| Hose system*                                | Functional test                 | —         | X       |
| <b>Height-adjustable base support units</b> |                                 |           |         |
| Surface damage                              | Visual inspection               | X         | X       |
| Height adjustment mechanism                 | Functional test                 | X         | X       |
| State of the gas spring*                    | Functional test                 | —         | X       |
| Joint bearing                               | Functional test                 | —         | X       |
| Height adjustment drive*                    | Lubrication                     | X         | X       |
| <b>Sub-pendant</b>                          |                                 |           |         |
| Surface damage                              | Visual inspection               | X         | X       |
| Lifting operation                           | Functional test                 | X         | X       |
| Adjustment of the anesthetic apparatus      | Functional test                 | X         | X       |

## 1.4 Replacement of parts subject to wear

### Repair Inspection intervals of the technical service

The components or functional systems mentioned below must be replaced by an authorized service technician at the specified intervals:

Legend:

- No measures required
- X Replacement recommended
- XX Compulsory replacement of parts

| Part subject to wear/functional system                                    | Every 5 | Every 10 |
|---|---------|----------|
| Gas hoses   | —       | XX       |
| Brake pads with levers and valves,<br>membrane keyboard, bellows cylinder | X<br>X  | —<br>—   |
| <b>Height-adjustable base support units</b>                               |         |          |
| Gas spring  | XX      | —        |
| Triggering system, control and limit switches                             | X       | —        |
| control valve   | X       | —        |
| membrane keyboard   | X       | —        |
| <b>Double pivot equipment adapter / lifting system</b>                    |         |          |
| Control and limit switches  | X       | —        |
| membrane keyboard   | X       | —        |