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
MAILLEFER INSTRUMENTS
MANUFACTURING S.à.r.l

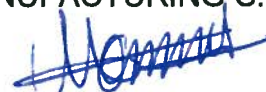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Re: Counterfeit DENTSPLY Products

Dear Sirs,

Our company belongs to the DENTSPLY group of companies ("DENTSPLY").

We discovered that Osakadental, Unit 04, 7/F, Bright Waytower NO.33 Mong KO Road, KOWLOON, HK has recently sold copies of our K-FILE COLORINOX[®], K-REAMER[®] and HEDSTROEM FILE COLORINOX[®] (see enclosed document describing the difference between our products and the counterfeits ones).

In addition, these products bear **false**:

- (i) **CE0459 mark**,
- (ii) **Reference numbers**, as the references indicated on these products do not match the size of the products as defined in the Dentsply Maillefer's internal procedures.
- (iii) **Lot numbers**, as the lot numbers indicated on these products do not match the Dentsply Maillefer's lot files **and**
- (iii) **Manufacturer's Name and address**, as Dentsply Maillefer never manufactured these devices.

Therefore, they are obviously not in conformity with the applicable legislation and may lead to vigilance issues.

We believe it is important to bring this issue to your attention and leave it to you to decide any measures you deem appropriate.

In any case, we remain at your disposal for any further information you may need.

We thank you in advance for your assistance and remain,

Yours sincerely,

Maillefer Instruments Holding Sàrl


Luc Etienne
QA/RA Director & Management Representative

Maillefer Instruments Holding Sàrl

Dominique Legros
General Manager CEO

Fake primary packaging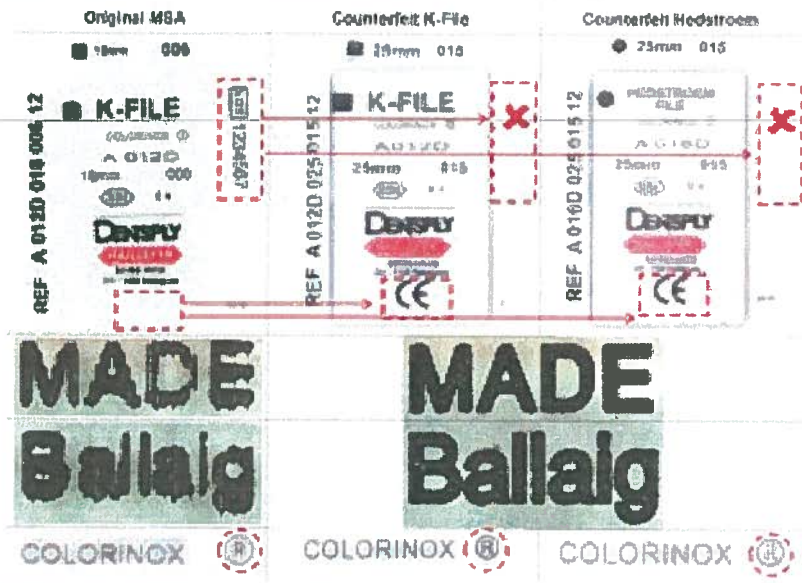

Counterfeit product has a ReOrder number ending in 12 and also carries a CE mark. DENTSPLY Mallefer does not produce such a product.

Counterfeit products do not have a lot number.

Counterfeit product labeling has a better quality compared to the original one.


Counterfeit product has the R marking different.

The lot number reference n°2350190 refers to Reamer instruments dedicated to surgical practice but not to root canal files.

Replacement of the word DENTSPLY by KOREA and the word «Ballaigues» by KOREA.

Fake versus original packaging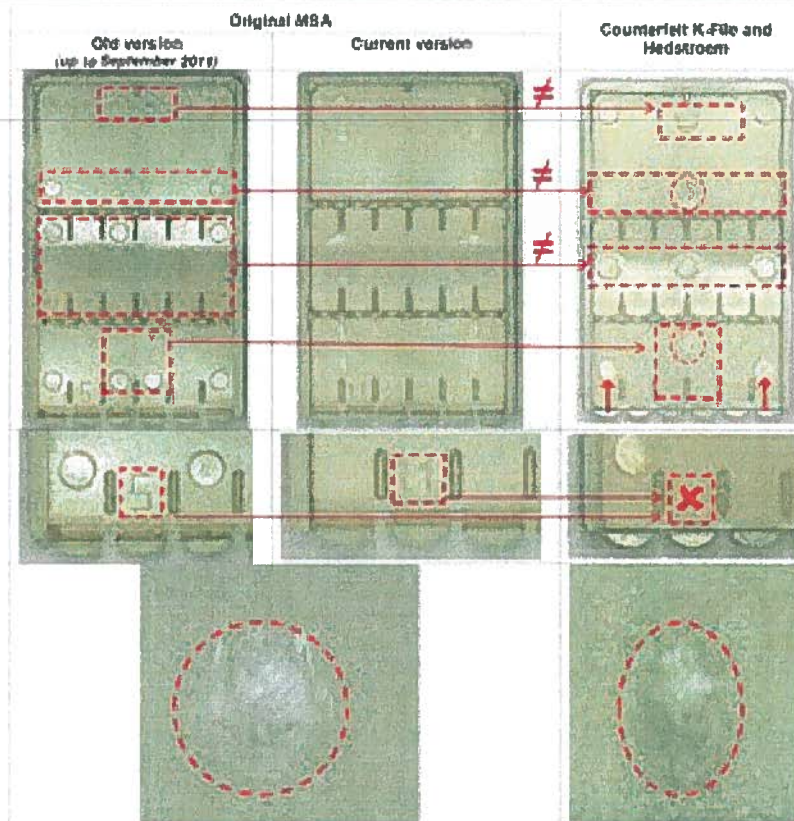


Ejection points are similar with the old Maillefer box version in terms of dimensions but their number and positions are different.

There is no identification number on the bottom inside the box.


The box aspect is different in terms of shaping.

Fake versus original product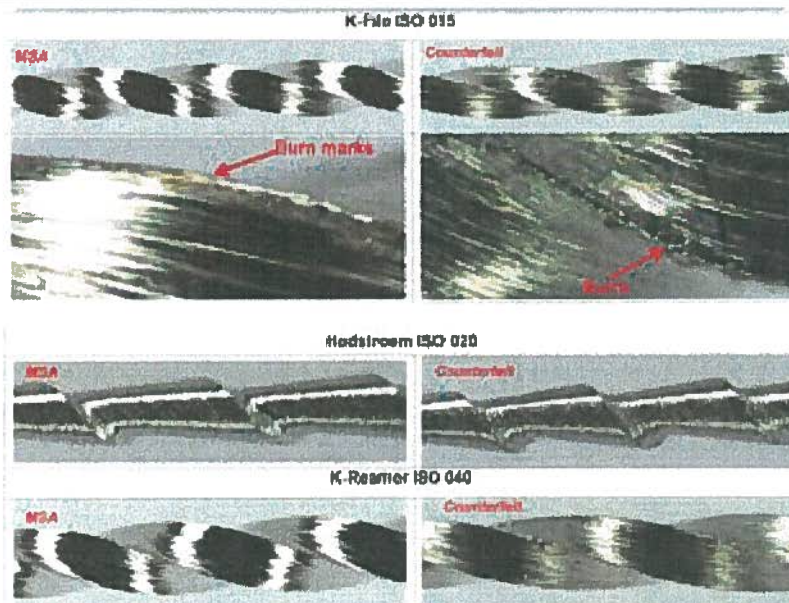


Counterfeit instruments had many burrs with punctual defect of twisting but they have the same face and flute number.

Fake versus original product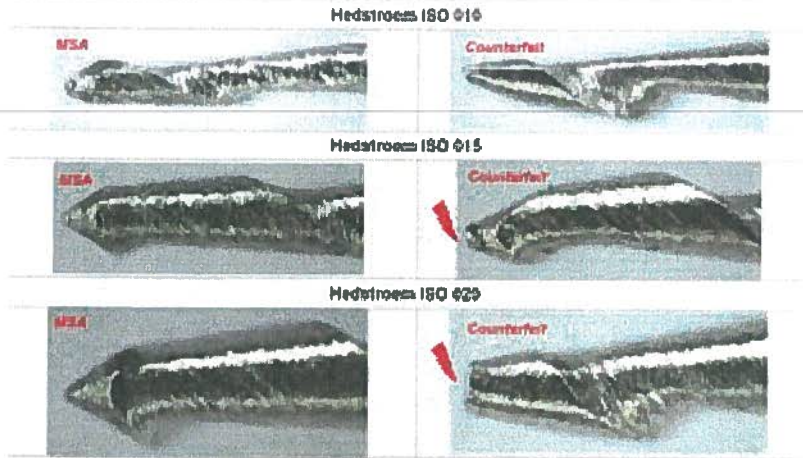


Counterfeit instruments show flattened tips with knock marks.

Fake versus primary packaging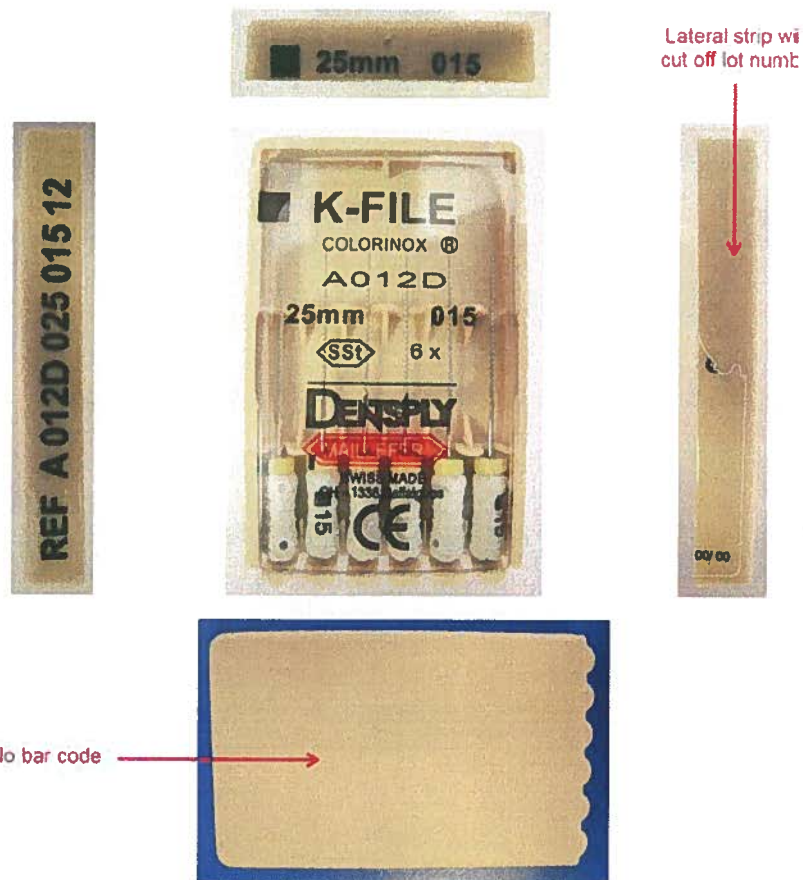

Counterfeit products do not have a bar code. DENTSPLY Maillefer does not produce such a product.


Fake versus original packaging (rear active part)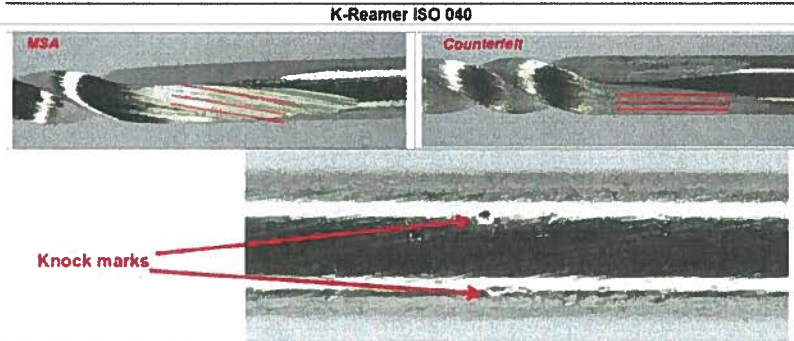

Counterfeit hedstroems have similar rear active part and similar grinding mark orientations.

Many marks are visible on the shaft.

Fake versus original product (handle)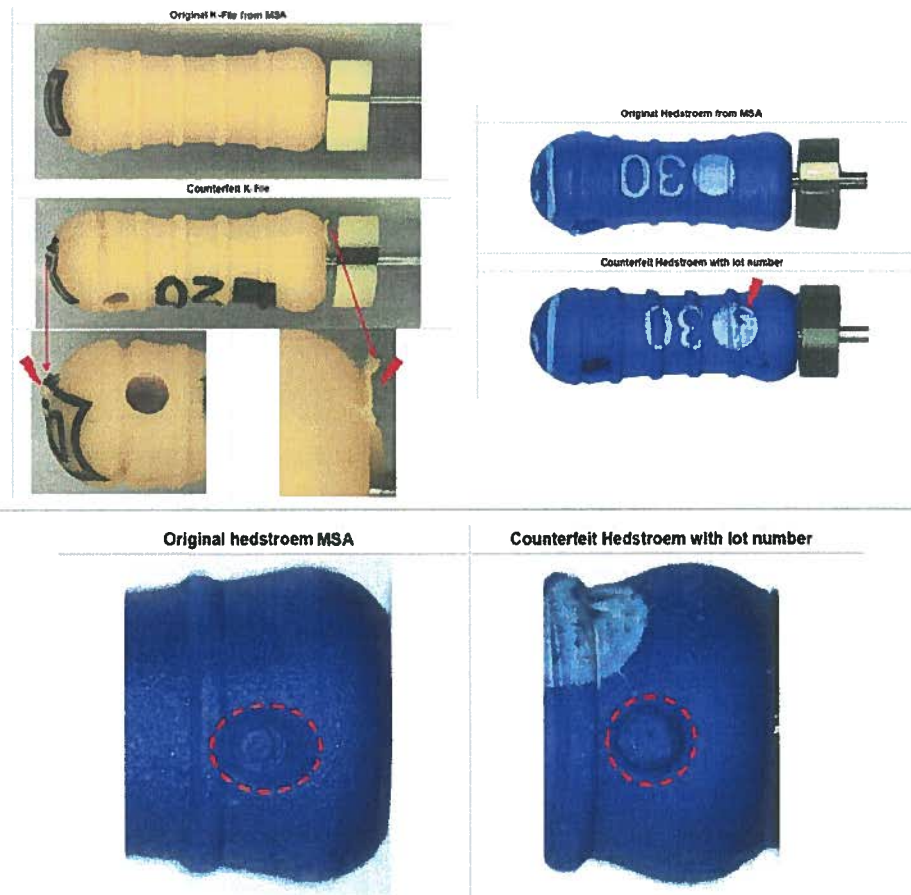


Counterfeit products had defect on the markings (oval injection point instead of round).


Counterfeit products had the same handle but with huge defect of injection.

Fake versus original product (silicon stops)

Counterfeit files without lot number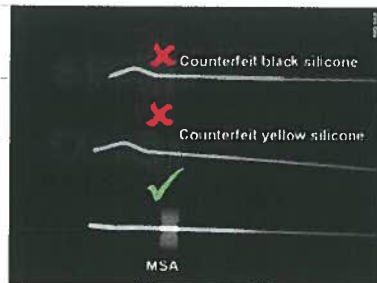

Counterfeit files with lot number

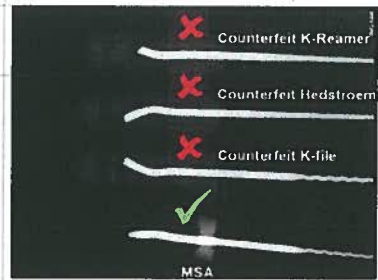Counterfeit product s silicon stops were not radio-opaque at all, whereas Maillefer s silicon stop could be easily seen on the radiograph.